

Open Fit Hearing Aid Tube Measurement Guide



(Actual Guides On Page #3)

For Determining Tube Length For All D.i.Y. Open Fit Hearing Aids

(888) 876-1113

**DIY
HEARING**

**Do-it-Yourself
Hearing Aids**

In order to deliver your hearing aid with a properly fitted tube design it is necessary to take a measurement of your ear so we have the correct tube length for your Open Fit Instrument.

The Sound Tip needs to fit into the upper portion of the ear canal.

Measure the right ear with the red side of the measuring card.

Measure the left ear with the blue side of the measuring card.


Place the measuring card snugly over the ear as shown in the picture below, with no gaps.

Position the 0, 1, 2, 3 markings by the ear canal as shown.

Ask someone to look into your ear canal while the measuring card is on your ear. Have them note the most appropriate size (1, 2, 3, 4) for your case.

A #1 tube would be required for the example shown where the #1 line is directly pointing toward the top portion of the entry to the ear canal.

Once you have determined the correct size tubing you will be asked for the measurement during the ordering process. Often one ear can be a different measurement from the other, so be sure to measure both ears.

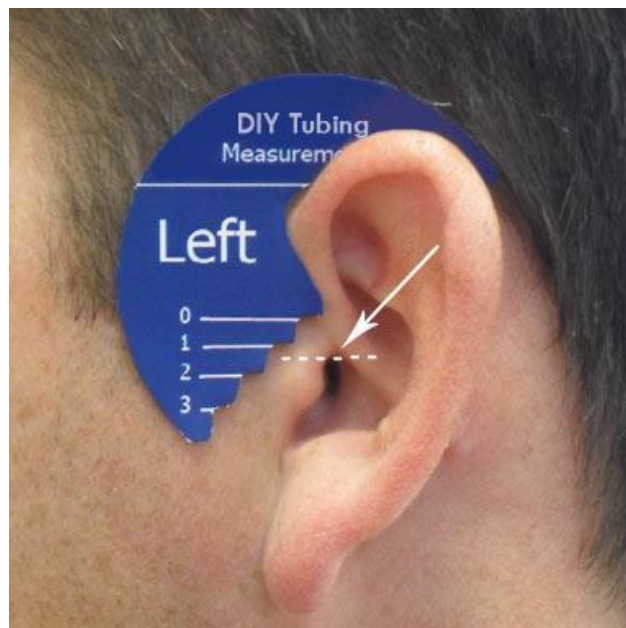

Above pictures are how the tubes are placed after your hearing aids are completed.

NOTE: It is important that this page is printed to 100% scale in order for this measuring card to be accurate.

Once printed, cut out the measuring card as shown on the patient's ear in picture above.

