


# Open Fit Hearing Aid Tube Measurement Guide

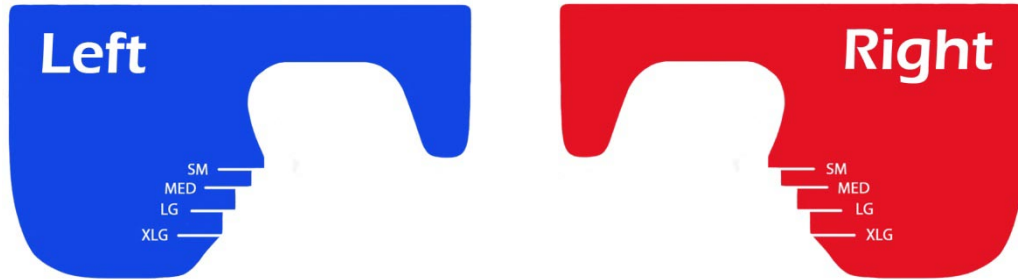

**For Determining Tube Length For All  
D.i.Y. Open Fit Hearing Aids**

**(888) 876-1113**

**DIY  
HEARING**

**Do-it-Yourself  
Hearing Aids**

In order to deliver your hearing aid with a properly fitted tube design it is necessary to take a measurement of your ear so we have the correct length for your Open Fit Instrument.

The Sound Tip needs to fit into the upper portion of the ear canal.

**Measure the right ear with the red side of the measuring card.**

**Measure the left ear with the blue side of the measuring card.**



Place the measuring card snugly over the ear as shown with no gaps.

Position the SM, MED, LG, XLG markings by the ear canal as shown.

Ask someone to look into your ear canal while the measuring card is on your ear. Have them note the most appropriate size (SM, MED, LG, XLG) for your case.

A medium tube would be required for the example shown where the Medium line is indirectly pointing at the top or aperture of the entry to the ear canal.

Once you have determined the correct size tubing you will be asked at checkout the size you have determined is correct for your D.i.Y. order.

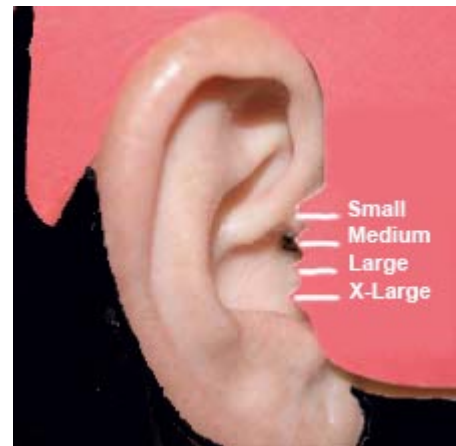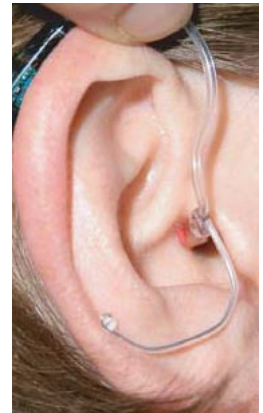

**NOTE:** It is important that this page is printed to 100% scale in order for this measuring card to be accurate.

Once printed, cut the card out and measure on the patient's ear as shown in picture above.

